

# BP Pump 2

## Non-Invasive Blood Pressure Simulator and Tester



## Technical Data

The BP Pump 2 provides dynamic blood-pressure simulations for testing adult and neonatal non-invasive blood pressure monitors, including both arm- and wrist-cuff types.

The analyzer features a preset mode for simulation of most patient conditions and the capability to program user-defined simulations. BP Pump 2 tests for leaks, measures static pressure, generates pressure, and tests overpressure valves. For improved testing versatility, the analyzer's recently upgraded waveform test suite includes additional physiological selections.

BP Pump 2 comes in two models: the standard BP Pump 2<sub>L</sub> and the BP Pump 2<sub>M</sub>. The BP Pump 2<sub>M</sub> features a high-accuracy pressure transducer to meet the EN1060-3 standard used widely in Europe for testing non-invasive blood pressure monitors. It also includes five-lead synchronized ECG simulations to test monitors that use ECG for motion rejection.

## Technical Specifications

### Pressure Generation

Static-Pressure Range: 0 mmHg to 400 mmHg  
 Difference Between Target Pressure and Actual Pressure:  $\pm 10$  mmHg from 100 mmHg to 400 mmHg with minimum volume of 300 cc  
 Internal Leak Rate:  $< 2$  mmHg per minute with minimum volume of 300 cc

### Four Respiratory Artifacts

3 spontaneous breathing; controlled ventilation

### 3 Adult Wrist-Cuff Simulations

Normal, Hyper, Hypo

### Pressure Source

Specified pressure generated from 0 mmHg to 400 mmHg in selectable increments of 1 mmHg

### Pressure Gauge

Static pressure measured from 0 mmHg to 400 mmHg at the pressure port

### Pressure Relief Rest

Test for the NIBPM pressure relief valve (0 mmHg to 400 mmHg) with display of peak pressure

## Key Features

### BP Pump 2<sub>L</sub> and BP Pump 2<sub>M</sub>

- Dynamic simulations for arm- and wrist-cuff monitors
- Recently upgraded waveform test suite with more physiological selections
- Internal pump for use in high- and low-pressure release verification, leak testing and pressure sourcing
- Preset mode for simulation of most patient conditions
- User-defined autosequences
- Internal cuff volume for basic device testing
- RS232 for computer control
- Compact, lightweight, and user friendly
- Respiratory artifacts, including spontaneous breathing and controlled ventilation
- Arrhythmia simulations, including premature atrial contractions #1 and #2, atrial fibrillation, and PVCs

### BP Pump 2<sub>M</sub> also includes:

- High-accuracy pressure transducer
- Five-lead synchronized ECG and arrhythmia simulation with blood pressure for both

### Neonate Internal Cuff Simulations

Internal neonate cuff; four standard neonate pressures

### Neonate Simulations

Cuff #1: Blood pressure: 35/15; heart rate: 120 BPM; pulse volume: 0.3

Cuff #2: Blood Pressure, 60/30; heart rate: 120 BPM; pulse volume: 0.3

Cuff #3: Blood pressure, 80/50; heart rate: 120 BPM; pulse volume: 0.3

Cuff #4: Blood pressure, 100/70; heart rate: 120 BPM; pulse volume: 0.3

### Normal Sinus Rhythm

BP and ECG: Healthy heart, weak pulse, mild exercise strenuous exercise, obese subject, geriatric subject, tachycardia, bradycardia irregular pulse

BP and ECG: Premature atrial contractions # 1, premature atrial contractions # 2, premature ventricular contractions, atrial fibrillation and PVCs

### User-Definable Simulations

User-definable systolic and diastolic values, along with heart rate and pulse volume

### Ranges

Systolic Pressure Range: 20 mmHG to 250 mmHG

Diastolic Pressure Range: 10 mmHG to 200 mmHG

Heart Rate: 30 BPM to 250 BPM

Pulse Volume: 0.1 cc to 2.4 cc in increments of 0.1 cc

### Simulation Parameters Performance

Max Pulse Volume: 2.4 cc

Max Heart Rate: 200 BPM at 2.4 cc pulse volume; 250 BPM at 1.2 cc pulse volume

Internal Neonatal Cuff Volume: 20 cc

Internal Adult Cuff Volume (Including NN Volume): 310 cc

Heart Rate Setting Accuracy:  $\pm 1$  BPM

Simulation Units: kPa and mmHg (user selectable)

### Pressure Leak Test

The pressure port is pressurized from 0 mmHg to 400 mmHg and keeps track of the pressure loss over time. Peak pressure and present pressure are displayed at all times; leak rate is displayed when it is available.

### Autosequences

Nine autosequences are provided for four tests and up to five simulations.

### Electrical ECG

Signals: RA, LA, RL, LL, V

Waveform: Lead II

Amplitude: 1 mV peak ( $\pm 10$  %)

Connections: Optional external ECG adapter physiological synchronization with NIBP

### Heart Rate for NIBP Simulations

Heart Rate Accuracy: + 1 BPM

Except for the Following:

- Patient condition weak pulse, tachycardia, obese, geriatric: + 1 % + 1 BPM
- Patient condition mild exercise: + 1.5 % + 1 BPM
- Patient condition strenuous exercise: + 3 % + 1 BPM

### Serial Port

Bidirectional RS232 port; baud rate of 9600 with no parity, one stop bit, and eight data bits

### Pressure Measurement

Pressure-Measurement Units: kPa, mmHg, cmH<sub>2</sub>O, cmH<sub>2</sub>O and psi (user selectable)

Range: 0 mmHg to + 400 mmHg

Resolution, BP Pump 2L (Basic Model):

0.1 kPa, 1 mmHg, 1 cmH<sub>2</sub>O, and 0.1 psi

Resolution, BP Pump 2M (High-Accuracy

Version): 0.01 kPa, 0.1 mmHg, 0.1 cmH<sub>2</sub>O,

0.1 in H<sub>2</sub>O, 0.01 psi

### Accuracy

- Basic Model (BP Pump 2L): 0 mmHg to 300 mmHg: + 0.5 % of reading + 1 mmHg; 301 mmHg to 400 mmHg: + 2 % of reading
- High-Accuracy Version (BP Pump 2M): < 0.8 mmHg (0.1 kPa) throughout range

### Parallel Port

25-pin female connector, with D-subminiature style and pinouts conforming to IBM PC printer port (unidirectional), HP and ASCII printers

### Sample Adult Arm-Cuff Simulation

Parameters (Standard)

Standard Set of Blood Pressures

BP #1: Blood pressure: 120/80 (93); heart

rate: 80; pulse volume: 0.68 cc

BP #2: Blood pressure: 150/100 (116); heart

rate: 80; pulse volume: 0.65 cc

BP #3: Blood pressure: 200/150 (166); heart

rate: 80; pulse volume: 0.6 cc

BP #4: Blood pressure: 255/195 (215); heart

rate: 80; pulse volume: 0.55 cc

BP #5: Blood pressure: 60/30 (40); heart

rate: 80; pulse volume: 0.75 cc

BP #6: Blood pressure: 80/50 (60); heart

rate: 80; pulse volume: 0.7 cc

BP #7: Blood Pressure: 100/65 (76); heart

rate: 80; pulse volume: 0.69 cc

### Patient Condition Simulations

Healthy Heart: Blood pressure: 120/80 mmHg

(93 MAP); heart rate: 75 BPM; pulse volume: 0.7 cc

Weak Pulse: Blood pressure: 110/80 (90);

heart rate: 95 BPM; pulse volume: 0.3 cc

Mild Exercise #1: Blood pressure: 140/90

(106); heart rate: 120 BPM; pulse

volume: 1.1 cc

Strenuous Exercise #2: Blood pressure:

140/90 (106); heart rate: 162 BPM;

pulse volume: 1.4

Obese Subject: Blood pressure: 120/80 (93);

heart rate: 90 BPM; pulse volume: 0.4 cc

Geriatric Subject: Blood pressure: 150/110

(12); heart rate: 95 BPM; pulse volume: 0.4 cc

Tachycardia: Blood pressure: 120/105 (110);

heart rate: 130 BPM pulse volume: 0.3 cc

Bradycardia: Blood pressure: 120/60; heart

rate: 45 BPM; pulse volume: 1.1 cc

### Arrhythmia Simulations

Premature Atrial Cont. #1: Blood pressure:

138/53 mmHg (81 MAP); heart rate: 80 BPM;

pulse volume: varies

Premature Atrial Cont. #2: Blood pressure:

144/64 (90); heart rate: 83 BPM; pulse

volume: varies

Premature Ventricular Cont.: Blood pressure:

118/61 (80); heart rate: 83 BPM; pulse

volume: varies

Atrial Fib and PVCs: Blood pressure: 139/72

(94); heart rate: 91 BPM; pulse volume: varies

### Respiratory Artifacts

Spontaneous Breathing #1: Blood pressure:

138/65 mmHg (89 MAP); heart rate: 104

BPM; pulse volume: varies

Spontaneous Breathing #2: Blood pressure

149/65 (93); heart rate: 105 BPM; pulse

volume: varies

Spontaneous Breathing #3: Blood pressure:

112/47 (68); heart rate: 86 BPM; pulse

volume: varies

### Controlled Ventilation

Blood Pressure: 132/44 (73); heart rate:

98 BPM; pulse volume: varies

### Wrist Simulations

Simulation #1: Blood pressure 120/80 (93);

heart rate: 80 BPM; pulse volume: 0.5 cc

Simulation #2: Blood pressure 160/100 (120);

heart rate: 80 BPM; pulse volume: 0.5 cc

Simulation #3: Blood pressure: 80/55 (63);

heart rate: 80 BPM; pulse volume: 0.5 cc

### Temperature

Operating: 15 °C to 40 °C

Storage: -20 °C to 65 °C

### Relative Humidity

90 ° max

### Display

Bright, large 4-line x 40-character alphanumeric display with back lighting

### Dimensions

10 in L x 10 in W x 5 in H

(25.4 cm x 25.4 cm x 12.7 cm)

### Weight

7.5 lb (3.4 kg)

## Ordering Information

### Model

#### BP Pump 2<sub>L</sub> (standard pressure transducer)

2249036: BPPUMP2L-US120V

2394895: BPPUMP2L-AUS250V

2394901: BPPUMP2L-DEN250V

2394912: BPPUMP2L-SHK250V

2394920: BPPUMP2L-ISR250V

2394935: BPPUMP2L-ITAL250V

2394947: BPPUMP2L-IND250V

2394958: BPPUMP2L-SWZ250V

2394964: BPPUMP2L-UK250V

#### BP Pump 2<sub>M</sub> (high-accuracy pressure transducer)

2249049: BPPUMP2M-US120V

2394973: BPPUMP2M-AUS250V

2394986: BPPUMP2M-DEN250V

2394999: BPPUMP2M-SHK0250V

2395003: BPPUMP2M-ISR250V

2395015: BPPUMP2M-ITAL250V

2395026: BPPUMP2M-IND250V

2395032: BPPUMP2M-SWZ250V

2395044: BPPUMP2M-UK250V

### Standard Accessories

2391882: Accessory kit (tubings and fittings)

N/A User/service manual

N/A Power cord (country specific)

### Optional Accessories

2222822: Soft-sided vinyl carrying case

2391894: ECG adapter block (allows simulation of 5-lead ECG waveforms)

2238072: Parallel printer cable, D25M-C36M

2248899: Printer, Seiko DPU-414-30B, 120 V power supply

2399531: Printer, Seiko DPU-414-30B, 200 V power supply

2235375: Printer, 120 V power supply

2235382: Printer, 220 V power supply

2248737: Printer paper (7 rolls min)

2238659: Serial cable, D9M-D9F

2392381: Adult cuff mandrel spacer block (three required)

2392370: Adult cuff mandrel end block (two required)

2392328: Neonatal/external cuff mandrel (truncated plastic cylinder diameters: 7.6, 10, and 14 cm)

2391875: Wrist cuff mandrel (adult)

### About Fluke Biomedical

Fluke Biomedical is the world's leading manufacturer of quality biomedical test and simulation products. In addition, Fluke Biomedical provides the latest medical imaging and oncology quality-assurance solutions for regulatory compliance.

Today, biomedical personnel must meet the increasing regulatory pressures, higher quality standards, and rapid technological growth, while performing their work faster and more efficiently than ever. Fluke Biomedical provides a diverse range of software and hardware tools to meet today's challenges.

### Fluke Biomedical Regulatory Commitment

As a medical device manufacturer, we recognize and follow certain quality standards and certifications when developing our products. We are ISO 9001 certified and our products are:

- FDA Compliant
- CE Certified, where required
- NIST Traceable and Calibrated
- UL, CSA, ETL Certified, where required

# FLUKE®

## Biomedical

### Fluke Biomedical.

*Better products. More choices. One company.*

#### Fluke Biomedical

PO Box 9090, Everett, WA 98206-9090 USA

#### Fluke Biomedical Europe AS

Vegamot 8, N-7048 Trondheim, Norway

#### For more information, contact us:

In the U.S.A. (800) 648-7952 or

Fax (425) 446-5629

In Europe/M-East/Africa +47 73954700 or

Fax +47 73954701

From other countries +1 (425) 347-6100 or

Fax +1 (425) 446-5629

Email: [sales@flukebiomedical.com](mailto:sales@flukebiomedical.com)

Web access: <http://www.flukebiomedical.com>

©2006 Fluke Biomedical. All rights reserved.

Specifications subject to change without notice.

Printed in U.S.A.

L1T060